

INSTRUCTIONS TO DISPOSE OF RED SHARPS BIN

Please follow the steps below to replace and dispose of the red sharps bin:

- 1 BIN ALMOST FULL** - the **HealthBeacon** will let you know when the red inner sharps bin is almost full and will automatically send you a new one.
- 2 BIN FULL** - when the “Bin Full” message appears on the screen, it’s time to replace your red sharps bin.
 - **Open** the side door and remove the full red sharps bin.
 - **Lock** the red sharps bin by sliding the top lid shut.
 - **Slide** the new replacement bin inside the HealthBeacon, ensuring that the door on the white lid is open and on the same side as the front of the HealthBeacon unit.



- 3 DO NOT RETURN THE RED SHARPS BIN IN THE MAIL**

Please dispose of the red sharps bin according to your state guidelines. More information regarding state guidelines can be found at:

www.SafeNeedleDisposal.org

